

Application Card

How to Use

ConvaTec AllKare® Protective Barrier Wipe*

Non-water soluble barrier that resists breakdown and “breathes” for greater comfort. Clinically tested for safety and efficacy with Stomahesive® and Durahesive® skin barriers.



Thin, comfortable film that helps protect against skin stripping and irritation caused by the application and removal of adhesives such as ostomy skin barriers and tapes.

Application Under an Ostomy Appliance

- Remove the skin barrier/pouch you are currently wearing.
- Thoroughly clean and dry skin (see reverse side for instructions on using AllKare® Adhesive Remover as part of cleansing routine).
- Apply AllKare® Protective Barrier Wipe and allow it to dry completely; do not touch.
- Rinse the area well with clear water and dry thoroughly.
- Apply the new skin barrier/pouch.

Additional Help for Irritated Skin

- Remove the skin barrier/pouch you are currently wearing.
- Thoroughly clean and dry skin
- Apply a light dusting of ConvaTec's Stomahesive® Powder to the skin surrounding the stoma, blow off any excess.
- Pat the powder with AllKare® Protective Barrier Wipe, making sure that all of the powder is wet (use two wipes if needed).
- Allow the area to dry thoroughly. A blow dryer set on cool may be used. The powder will show a shiny coat.
- Apply the new pouch.

How to Use

ConvaTec AllKare® Adhesive Remover Wipe*

A non-irritating, formulation that provides two products in one: an adhesive remover and a skin emollient. Emollient-rich formula helps to leave skin feeling supple, soft and smooth.



For the gentle removal of all adhesives, including skin barriers, tapes and hydrocolloid dressings.

For use in ostomy management

- Gently lift a corner of the skin barrier or tape.
- Wipe the adhesive remover towelette between the skin and the skin barrier or tape, gently peeling the skin barrier or tape back while continuing to wipe.
- Cleanse the skin around the stoma with mild soap that is free of oils and creams.
- Rinse the area well with clear water and dry thoroughly.
- Apply the new skin barrier/pouch.

Use an AllKare® Adhesive Remover Wipe on the skin around the stoma with each skin barrier change or as needed.

*See package insert for complete Instructions for Use.