



ESENTA™ STING FREE ADHESIVE REMOVER SPRAY

EASY TO USE: FOLLOW THE BELOW STEPS

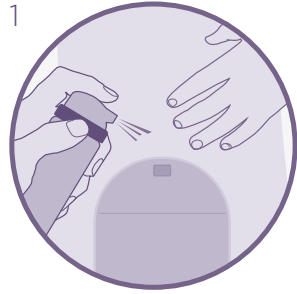


GETTING STARTED

Spray around the edge of the adhesive and wait a few seconds. You can spray at any angle.

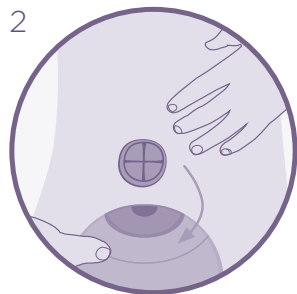

EASY REMOVAL

Gently peel adhesive away from the skin using additional ESENTA™ adhesive remover spray to ease removal, if necessary.

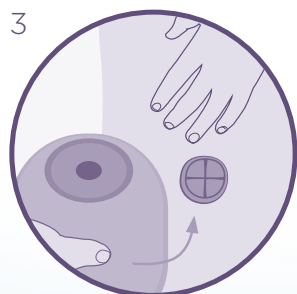


DRIES IN SECONDS

Once dried ESENTA™ Adhesive Remover will not affect adhesion of further dressings or appliances.




ESENTA™ STING FREE ADHESIVE REMOVER WIPES

EASY TO USE: FOLLOW THE BELOW STEPS

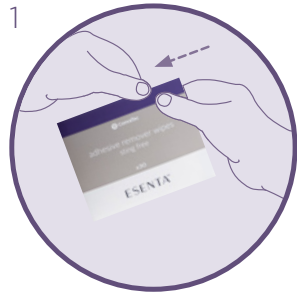


GETTING STARTED

Tear sachet to open and unfold wipe.

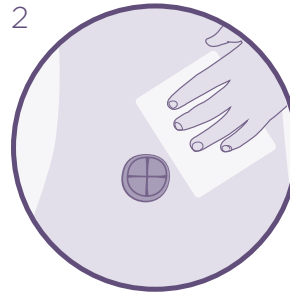

EASY REMOVAL

Rub the wipe over adhesive residues to remove.

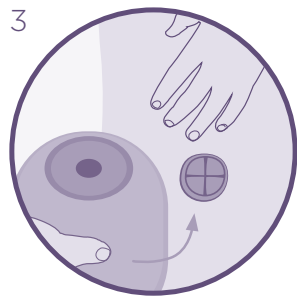

DRIES IN SECONDS

Once dried ESENTA™ Adhesive Remover will not affect adhesion of further dressings or appliances.



Learn more at convatec.com or contact us at 1-800-422-8811 or cic@convatec.com.

See package insert for complete instructions for use.

