Boys' User Guide Advance Plus™ Catheters

1. Wash your hands.
2. Fully fold back the packaging by peeling the clear side of the package.
3. Remove the cap from the protective tip.
4. Hold the gel reservoir in one hand. With the other hand, move the catheter forward until the tip of the catheter moves up into the protective tip.

Hint: Don't squeeze the gel reservoir too hard. Also be sure the tip of the catheter doesn't poke out the top of the protective tip.

5. Hold your penis. Pull back the foreskin (if present) and clean the tip of your penis with mild, unscented soap or a non-alcoholic wet wipe.

Hint: Don't squeeze your penis too hard. This can block the opening and make it difficult to insert the catheter.

6. Holding the gel reservoir gently, insert the protective tip until the base comes in contact with the urethral opening. Keep the protective tip in place. Relax the foreskin (if present).

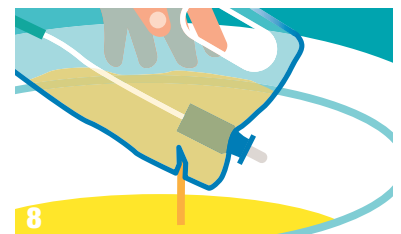
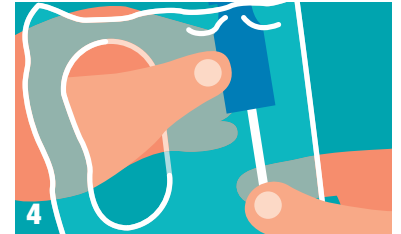
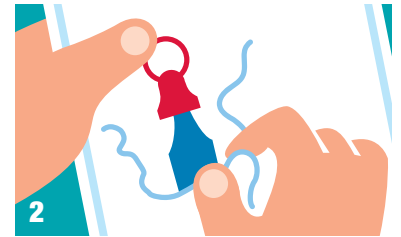
7. The gel reservoir should be held gently in place until the catheter has passed through the urethra and into the bladder.

Hint: As you advance the catheter and the pouch begins to gather, if you want to pull back the pouch, hold the catheter firmly through the gel reservoir. This action will help prevent the catheter from slipping out of the urethra before you gently pull back on the pouch. Continue to insert the catheter forward until the urine starts to flow. Once urine has stopped flowing, slowly withdraw the catheter.

8. To empty before disposing, tear at notch to drain urine.


9. Throw the catheter and pouch/bag away in the garbage.

10. Wash your hands again with soap and water.

Advance Plus
Intermittent Catheter

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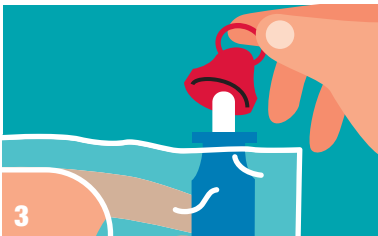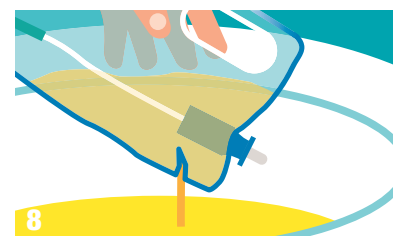
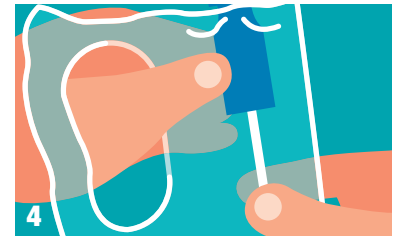
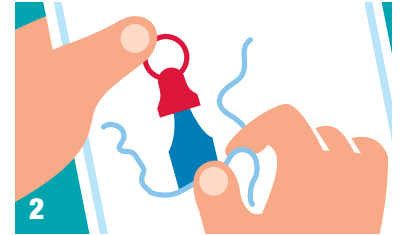
Advance Plus

Sonda Intermitente

WARNING: To help reduce the potential for infection and/or other complications, do not reuse.

WARNING: If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

- Ready-to-use, pre-lubricated, single-use closed system intermittent catheter
- Protective tip and collection bag supports no-touch insertion technique
- Gel reservoir
- 2 smooth catheter eyelets
- Color-coded funnel
- 1500 ml collection pouch

This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and pediatric patients who need to drain urine from the bladder.

PRECAUTION: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False Passage
- Urethritis — inflammation of the urethra
- Prostatitis — inflammation of the prostate gland
- Epididymitis — inflammation of the epididymis (testicle tube)

PRECAUTION: Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization frequency varies by person, the recommended frequency of your catheterization should be provided by your healthcare practitioner. For any other questions about your catheterization, please contact your healthcare practitioner.

NOTE: Store boxes in a flat position and at normal room temperature.

Rx Only. Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product.

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